

ORDER OF DRAW CHART

LID COLOUR		COLLECTION TUBE	INVERT TO MIX
Aerobic	Grey flip off lid	Blood cultures	8 - 10 times
Anaerobic	Purple flip off lid		
	Light blue	Citrate tube	3 - 4 times
	Bright yellow	CPD tube	8 - 10 times
	Gold	SST™II gel tube	8 - 10 times
	Red	Plain serum tube (with clot activator)	8 - 10 times
	Green	Heparin tube	8 - 10 times
	Purple / Pink	EDTA tube	8 - 10 times
	Grey	Fluoride oxalate and fluoride EDTA tubes	8 - 10 times
	Navy Blue	Trace element	8 - 10 times
Nil tube negative control	Grey	Quantiferon-TB Gold	Shake 10 times
TB1 Antigens	Green		
TB2 Antigens	Yellow		
Mitogen tube positive control	Purple		